




Symphony®

BREASTPUMP

USER INSTRUCTIONS




Pictured above....Symphony with Double Pump Kit

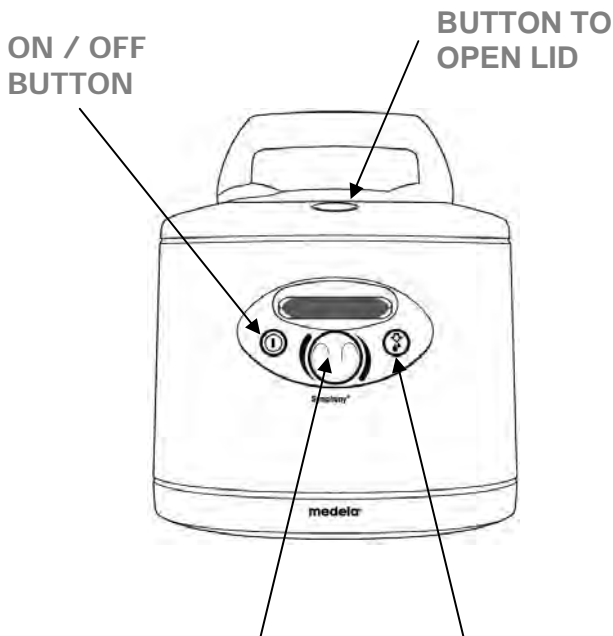

1. Press button near handle to open lid of Symphony Breastpump. **Ensure the Protective Membrane & Membrane Cap is secured in place.** One (1) Protective Membrane and Membrane Cap is required for single pumping and two (2) Protective Membranes and Membrane Caps are required for double pumping.
2. Insert one end of silicone tubing into Membrane Cap & close lid.
3. Insert other end of silicone tubing into the back of the Breastshield, which will be attached to the bottle. Make sure there is a valve & membrane attached to the bottom of the breastshield.
4. Press ON / OFF button to turn Symphony on.
5. The Symphony Breastpump will start in Stimulation Phase. Adjusting the One Knob Control in this phase alters only the vacuum level (speed remains consistent 120 cycles/min)
6. After 2 minutes in Stimulation Phase, Symphony will automatically switch into the Expression Phase. If the mother's milk let down reflex occurs **BEFORE** the 2-minute stimulation has elapsed, press the Let Down button to start the Expression Phase. Turn the One Knob Control to increase the vacuum to a comfortable level. As you are in the Expression Phase, turn the One Knob Control clockwise to increase the vacuum to your Maximum Comfort Level (you will see blocks increase on the LCD screen as you do this). Once Maximum Comfort Level has been reached, then turn the One Knob Control anti clockwise to reduce one block on the LCD screen. This will be the level for your most efficient expression.
7. The One Knob Control adjusts the vacuum levels and the cycles per minute in the Expression Phase, ie. higher vacuum levels will result in lower cycles per minute and higher cycles per minute will result in lower vacuum levels.



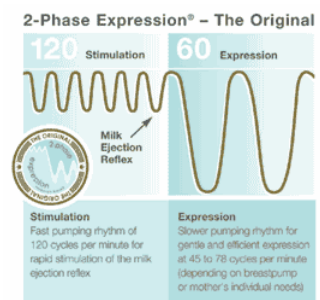




ONE KNOB CONTROL LET DOWN BUTTON



Breastmilk is a bodily fluid! For hygiene reasons, every customer must purchase their own milk collection kit.

These kits are for single patient use only and **should not** be shared between mothers

Cleaning Instructions for in Home Use

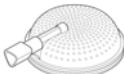Part	Pre-Cleaning Removal of dirt and dust with detergent		Disinfection Same Patient Use			
			Brush	Wash	Soak Eg. Miltons, tablets	
CLEAN AFTER EACH USE						
<i>Breastshield</i>	Yes	Yes	Yes	Yes	Yes	
<i>Valve</i>	Yes	Yes	Yes	Yes	Yes	
<i>Membrane</i>	Yes	Yes	Yes	Yes	Yes	
<i>Bottle</i>	Yes	Yes	Yes	Yes	Yes	
CLEAN AS REQUIRED						
<i>Silicone Tubing with Polygon Adapter</i>	Yes	Yes	Yes	Yes	Yes	
<i>Protective Membrane</i>	Yes	Yes	Yes	Yes	Yes	
<i>Membrane Cap</i>	Yes	Yes	Yes	Yes	Yes	

CAUTION: MICROWAVING WILL DESTROY THESE PRODUCTS

Fisher & Paykel Healthcare do not take responsibility for Medela Breastpumps performance if cleaned by methods not listed.