



## Ameda Elite® and HygieniKit® Operating & Cleaning Instructions


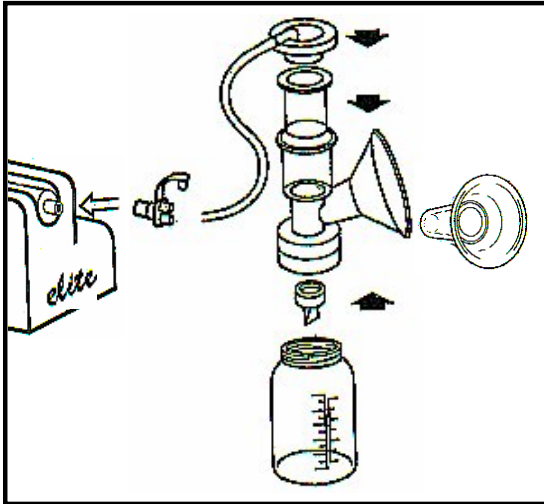


### Assembling the Ameda HygieniKit®

#### To Single Pump

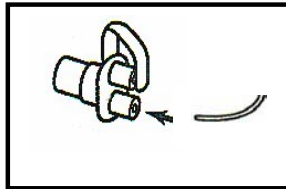

#### To Double Pump

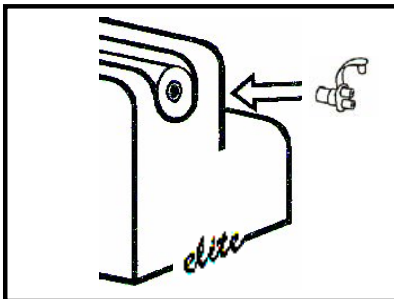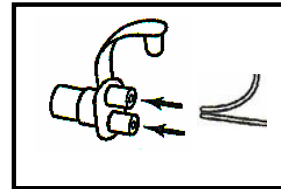

### Operating the Ameda Elite® Electric Breast Pump

4. Assemble the Ameda HygieniKit© Milk Collection system as per the instructions included with the product.
5. Insert the tubing adaptor on the HygieniKit© Milk Collection System into the piston cylinder opening.
6. Centre your nipple(s) into the breast flange(s) of the HygieniKit© Milk Collection System.

### Starting the Ameda Elite® Electric Breast Pump

3. Slowly, turn the vacuum dial to the right to the highest position most comfortable for you. Note: High suction does not necessarily produce more milk.

### Setting the Cycle Speed


1. Turn the cycle dial all the way to the right to help trigger the let – down reflex. Once your milk begins to flow, turn the dial back to the left. As your milk begins to slow down, repeat step 5 to trigger more let-downs.
2. When finished pumping, turn the vacuum dial all the way to the left to turn pump off.



**\*\*Important\*\***

**Breast pump MUST be turned  
off at power point when not in use**

## Cleaning the Ameda HygieniKit®

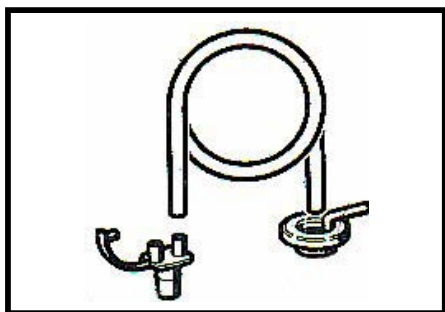


**After use,**

**Wipe over the tubing adaptor, tubing and adaptor cap.**

**Do not wash.**

**Does not require cleaning as it does not come in contact with mothers' milk.**

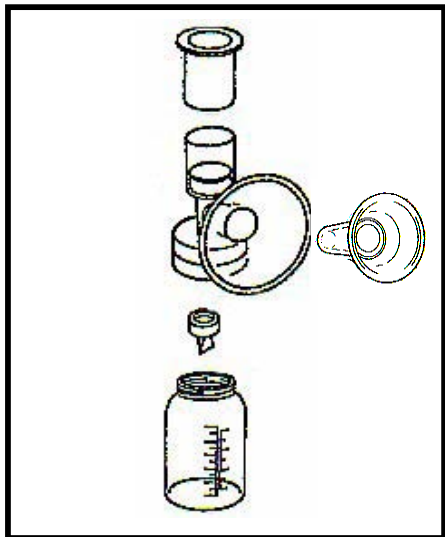

**Wash gently :**

**Silicone diaphragm, pump body, valve, bottle and reducing insert,**

**in warm soapy water. Soak valve if required.**

**Rinse gently. Do not use bottle brush.**

**Place items for disinfection/sterilization according to hospital protocol.**

### **IMPORTANT :**

**Sterilization instructions :**

#### **Hospital**

**For Eco HygieniKit (polypropylene) :**

**134 deg C for up to 5 mins**


**For HygieniKits (polycarbonate) :**

**121 deg C for 15 mins**

#### **Home**

**Boil for 20 mins or wash in warm soapy water.**


**Air dry before assembly.**

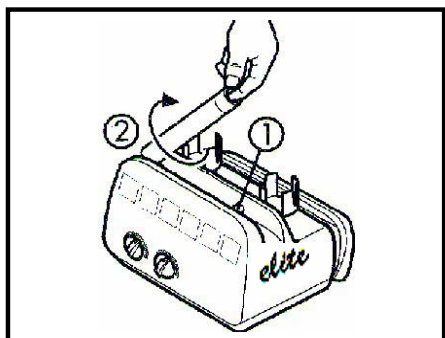

### **Cleaning the Elite :**

**Wipe Elite with hospital grade disinfectant with a damp cloth. Remove the cylinder by placing the index finger into the tube and by turning clockwise releasing it from the attachment.**

**Do not use abrasives or solvents.**

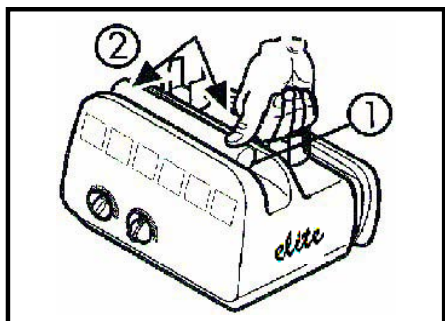

**Reinstall the cylinder as shown in the picture, In attachment points ① and ②. Press and click in place.**

220/240 V ~, 50/60 Hz  
110/120 V ~, 50/60 Hz  
30VA

Technical data  
Mains voltage, Frequency  
Power/current

CE 0123  
CSA NRTL/C Mark

Classification  
Safety tests

0 - max. 350 mbar

Vacuum range